



## Urine Specimen Collection Guide – No Preservation

For *Chlamydia trachomatis*, *Neisseria gonorrhoeae*, and/or *Trichomonas vaginalis*

### Collection for Male and Female Urine Specimens



*Patient should not have urinated for at least 1 hour prior to specimen collection.*

Direct patient to provide **first-catch urine** (Approximately 20-30 mL of initial urine stream) into urine collection cup free of any preservatives. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity.

**Female patients should not cleanse labial area prior to providing specimen.**

### Specimen Transport and Storage

Urine sample in primary collection containers must be transported to lab at 2°C to 30°C **within 24 hours of collection.**



### **For Test Collection Supplies Please Contact:**

Laboratory Alliance of Central New York, LLC  
113 Innovation Lane, Liverpool, NY 13088  
Ph: (315) 410-7067 Fax: (315) 410-7062  
[www.laboratoryalliance.com](http://www.laboratoryalliance.com)